


Clear reports you can view,
print, send and keep.

With Accu-Chek Smart Pix everything becomes
simple and clear in an instant.

You can print out your charts for your own use and
email or fax them to your healthcare professional.

View your results in simple formats.

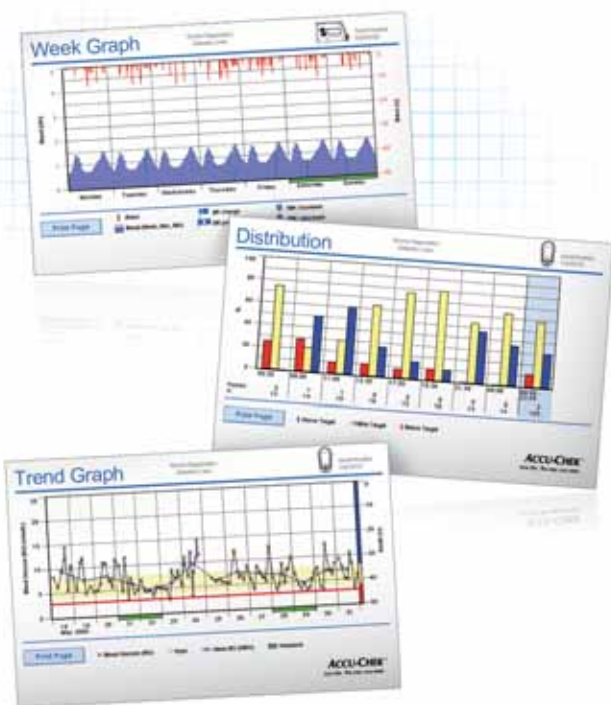

Key features include:

- Overview reports summarising your blood glucose results over a defined period.
- Daily, weekly, monthly charts, and trend graphs.
- Easily spot out-of-range readings.
- Review how your results behave day by day.
- Track patterns that you might not have noticed otherwise.

Order your Accu-Chek Smart Pix today.

To place your order, freephone
the Customer Careline on

Special Offer*
£14.99
€19.59

 **0800 701 000** (UK)

 **1 800 709 600** (ROI)

and quote C1009.

*Offer ends 30.06.09.

or for more information visit

www.accu-chek.co.uk/smartpix

- Accu-Chek Smart Pix works straight away.
- Connects via USB to your computer.
- Your results are automatically displayed in a choice of formats.
- Compatible with all Accu-Chek blood glucose meters and Accu-Chek insulin pumps.

Minimum Requirements: Computer with a free USB hub. Operating System: Microsoft Windows 2000 (service pack 4 / SP4) XP and Vista. Microsoft Internet Explorer IE 5, IE 6, IE 7 or Mozilla Firefox 1 and 2.

ACCUCHEK, ACCUCHEK SMART PIX and ACCUCHEK AVIVA NANO are trademarks of Roche. © 2009 Roche Diagnostics Limited.

Roche Diagnostics Limited, Charles Avenue, Burgess Hill, RH15 9RY
Company registration number: 571546

PM005521262001



ACCUCHEK®

ACCUCHEK® Smart Pix

Roche

Diabetes Management

Accu-Chek® Smart Pix
Get a clear data picture - easily



ACCUCHEK®

Accu-Chek Smart Pix.

The easy way to understand your blood glucose results.

Managing your diabetes isn't always easy. You need to carefully manually record and analyse your results.

Help is at hand, Accu-Chek Smart Pix turns manual log books into a thing of the past.



Smart, simple and fast.

Accu-Chek Smart Pix works very simply. It reads the results from your blood glucose meter or insulin pump and displays them in easy-to-understand charts on your computer.

Accu-Chek Smart Pix is compatible with all Accu-Chek blood glucose meters and Accu-Chek insulin pumps.



View your results in 3 simple steps.

1 Plug

Plug Accu-Chek Smart Pix into a USB port on your computer. Under "My Computer", click the **start.htm** icon in the Smart Pix drive to launch the Smart Pix software.



2 Place

Place your Accu-Chek blood glucose meter or Accu-Chek insulin pump in front of your Accu-Chek Smart Pix.

Press the two buttons on your device to activate the results transfer and click **Read Device** on your computer screen.



3 Play

Your blood glucose results will show on your computer screen. Click the **Create Reports** option at the top of the screen and select the charts you want to view to ease the discovery of trends that may be critical to your health.



Keep your Accu-Chek Smart Pix version up-to-date.

The technology behind Accu-Chek Smart Pix is constantly developing. Periodically, we release complimentary software updates.

Our latest update, Version 3 is FREE and easy to download. Upgrading your Accu-Chek Smart Pix to Version 3 will ensure you benefit from the most advanced functions including compatibility with the new Accu-Chek Aviva Nano meters.



Simply:

• Plug your Accu-Chek Smart Pix into a USB port and click **Start.htm**.

• Log on to www.accu-chek.co.uk/upgradesmartpix, register, click on the **download update** option and save onto your Smart Pix drive.

• Click on **Start.htm**. Your Accu-Chek Smart Pix will flash for 20 seconds as the software upgrades.