

# Changing lancet drum

## To load drum



Remove cap. Insert new drum with **blue** ring end first. **(Contains 6 lancets.)** Snap drum into place. Replace cap.

## To change drum



When no white bars appear in display, last lancet is in use. Remove cap. Remove used drum (has **red** stripe).

*Used drums cannot be re-inserted, and can be safely disposed in household waste.*

**With Accu-Chek Multiclix,  
CHANGE DRUM ONLY EVERY 6 LANCETS.**

# Changing depth setting

## Set penetration depth



Select depth by turning dial. For beginners, select a lower setting (start with level 1 or 2).

## Proven least painful<sup>1</sup> compared to other finger prickers

**Safety Advice:** The Accu-Chek Multiclix finger pricker is intended for patient self-monitoring by an individual person only. It must not be used to collect blood in a multipatient setting as it does not incorporate any features to guard against cross infection.

1. Koubek RJ, Accu-Chek blood glucose monitoring systems and lancing devices are least painful. A comparative survey (2007). Data on file.

ACCU-CHEK and LIVE LIFE. THE WAY YOU WANT. are trademarks of Roche. © 2008 Roche Diagnostics Limited.

Roche Diagnostics Limited,  
Charles Avenue,  
Burgess Hill, RH15 9RY  
Company registration  
number: 571546  
[www.accu-chek.co.uk](http://www.accu-chek.co.uk)



**ACCU-CHEK®**  
Live life. The way you want.

# Faster, easier and safer blood sampling



**ACCU-CHEK® Multiclix**



**ACCU-CHEK®**  
Live life. The way you want.

# Easy everyday use

## 1 **PRESS** to prime

Press priming button down like a ballpoint pen.



*The transparent release button will become yellow.*

## 2 **CLICK** to prick

Press lancing device against side of fingertip and press (yellow) release button.



## 3 **TWIST & BACK** for next lancet\*

Turn priming button in direction of arrow as far as it will go, and turn back again.



*The white bars on display show how many lancets are left.*

*It is recommended to always use a new lancet.*

# Half the steps Twice as easy

ACCU-CHEK® Multiclix lancing device is intended for personal use only

\* Finger pricks are less painful if a new lancet is used for each test